



# Customer Release Notes

eRAD PACS  
Version 8.0  
Server Build 31

Release Date June 11, 2021

## Table of Contents

<b>New Features and Enhancements</b> .....	3
1. Optimize worklist action device list .....	3
2. Add subsections to forward target configuration interface.....	3
3. Select All user groups avoids Everyone group.....	3
<b>Resolved Issues</b> .....	3
1. User edit color scheme color picker missing .....	3
2. Users without CD importing rights can import DICOM media from the GUI.....	3
3. Missing archive page saved lists on other lists page .....	3
4. Display of Is NOT filter is "Empty" .....	3
5. User Group edit page does not show all selected group IDs in batch edit mode.....	3
6. All dashlet types are available to users with no rights .....	4
7. Reject compound lists consisting of a compound list or a list from a different user .....	4
8. Adding a new Dicom device creates the device with all storage SOP classes disabled by default .....	4

## New Features and Enhancements

### 1. Optimize worklist action device list

The layout of the device targets list on the Forward and Stop Sending Updates action setup pages has been changed to optimize the available space.

### 2. Add subsections to forward target configuration interface

The device targets list on the user and group edit pages has been rearranged to group devices into sub-sections for easier identification.

### 3. Select All user groups avoids Everyone group

The Everyone group is a special group created by the system and does not support batch edit requests. As a result, the Everyone group is silently excluded when batch editing multiple groups on the Group Accounts page, whether it is selected or not.

## Resolved Issues

### 1. User edit color scheme color picker missing

The color selection panel in the Preferences section of the user account edit page was missing on the first display attempt. This has been resolved.

### 2. Users without CD importing rights can import DICOM media from the GUI

Menu items with no dedicated web page, such as importing media, failed to check permissions prior to invoking the request, allowing users without media import rights to import studies. This has been resolved.

### 3. Missing archive page saved lists on other lists page

Saved Archive page lists have been added as a group to the Other Lists page.

### 4. Display of Is NOT filter is "Empty"

Applying negative search criteria to an enumerated value field, such as State, displayed the correct results but the criteria displayed on the page was incorrect. This has been resolved.

### 5. User Group edit page does not show all selected group IDs in batch edit mode

The batch edit function on the Group Accounts page failed to collect all the selected groups IDs, editing only the first selected group. This has been resolved.

**6. All dashlet types are available to users with no rights**

Permissions have been assigned to some dashlets. Users must have administrator rights to create and view dashlets showing system statistics. Users must have support rights to create and view dashlets that can edit database records or issue custom database queries.

**7. Reject compound lists consisting of a compound list or a list from a different user**

Corrected exceptions to the rules in which compound worklist filters cannot be created from compound filters, and all filters in a compound filter must be owned by the user creating the compound filter.

**8. Adding a new Dicom device creates the device with all storage SOP classes disabled by default**

When adding a device, the list of supported SOP classes defaults to the system default list but failure to check for an empty list in addition to an uninitialized list caused the default to support no SOP classes. This has been resolved.