



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 24

Release Date March 5, 2021

Table of Contents

New Features and Enhancements.....	3
1. Show VR field on the customize labels page without support rights.....	3
2. Move study tool	3
Resolved Issues.....	3
1. Higher level default worklist should appear after deleting a saved default worklist	3
2. Clicking Confirm or Cancel button does not close the filter window	3
3. Copy from worklist to folder or worklist doesn't work from CWL servers	3
4. Error processing a login call for a locked user.....	3
5. Group actions ignored if the length of the group id is 20 characters.....	3
6. Initials matching broken.....	4

New Features and Enhancements

1. Show VR field on the customize labels page without support rights

The database field value representations (VRs) are displayed on the Admin/System/Customize Labels pages for admin users.

2. Move study tool

Users with admin or study edit permission can move a study from one hub server to another from the topmost worklist server. The tool is available in the worklist's toolbar or floating toolbar. The selected studies appear on a curtain page from which the user selects the target hub server. Orders and shortcuts, if selected, will not be moved. A notification is sent to the user when the move starts and again when the move completes or fails. Check the messaging dashlet for notifications if they are not visible from the notification indicator.

Resolved Issues

1. Higher level default worklist should appear after deleting a saved default worklist

After deleting a default worklist label from the list menu, the group or system default list should appear but did not because the list was not refreshed. This has been resolved.

2. Clicking Confirm or Cancel button does not close the filter window

The button controls on some curtain pages failed to close the popup panel, leaving the popup panel visible after the operation was completed or canceled.

3. Copy from worklist to folder or worklist doesn't work from CWL servers

When replicating a study from the consolidation server, the target folder was created but the redirection request failed due to a call to a retired page. This has been resolved.

4. Error processing a login call for a locked user

Login requests from web services calls or when reading location was specified failed to check the lock state of the user account.

5. Group actions ignored if the length of the group id is 20 characters

Actions assigned to groups whose ID was 20 characters were ignored because the last character was truncated from the identifier saved in the action database table, meaning it didn't match any registered group when actions ran. This has been resolved.

6. Initials matching broken

The recent fix to handle double quotes in patient matching criteria fields introduced a parsing bug when patient matching including initial matching parameters. This has been resolved.