



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 43

Release Date December 10, 2021

Table of Contents

New Features and Enhancements.....	3
1. Add "Use default" option for user profile settings.....	3
2. Procedure codes cannot be mapped from Web Service orders to MWL.....	3
3. Rename <i>Select all</i> check box on DICOM devices page.....	3
Resolved Issues.....	3
1. Purge action with Purge Matching Objects doesn't work on Objectless CWL servers.....	3
2. MCS server creates media with empty <i>_report.pdf</i>	3
3. Editing Customize layout XML files returns an unexpected exception.....	4
4. Action owner is case insensitive	4

New Features and Enhancements

1. Add "Use default" option for user profile settings

The setting menu for each field listed on the copy profile page includes an option to reset the setting to its system default setting. Note that resetting a profile setting to the system default can cause the account to inherit a setting from the user's primary group settings, if the group explicitly overrides the system default. Additionally, this enhancement includes a summary mode option. When the Toggle Summary button (at the bottom of the curtain, next to the save and cancel buttons) is selected, or after the user clicks the Save button, the list of modified profile settings appears for review. You can save the changes or return to the editing mode to make further changes before saving them.

2. Procedure codes cannot be mapped from Web Service orders to MWL

Procedure code values submitted in the web services interface failed to include the coded entry data item when mapping the value to DICOM's requested procedure code sequence, causing some modality worklist clients to fail to recognize the procedure code. Refer to the EP Web Services Programmer's Manual for details.

3. Rename *Select all* check box on DICOM devices page

The label to select all SOP classes on the DICOM device configuration page was modified for clarity and consistency.

Resolved Issues

1. Purge action with Purge Matching Objects doesn't work on Objectless CWL servers

When setting up a purge action to delete select study objects from an object-less worklist server running in a mixed mode environment, the system could not identify the selected objects because it expected them in the local object table. As a result, it failed to remove the objects when the action ran. This has been resolved.

2. MCS server creates media with empty *_report.pdf*

When using a remote MCS server, if the user initiating the media creation job does not exist on the MCS server, an exception occurred and an empty report file was included on the media. Note that if the user initiating the job does not exist on the MCS server, the MCS uses the system default user account and its settings to create the media.

3. Editing Customize layout XML files returns an unexpected exception

The XML layout customization pages have not been updated to use v8 technology. As a result, some features might result in an exception when applied, such as previewing a customized template page.

4. Action owner is case insensitive

The user group fields in the action table were case insensitive while user and group IDs are case sensitive, causing two users with the same ID, except for case, to see and affect each other's assigned actions.