



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 27
Release Date April 15, 2021

Table of Contents

New Features and Enhancements.....	3
1. Version information change for PAAS.....	3
2. Change secure connection default to AUTO.....	3
3. Restore missing disk usage features	3
Resolved Issues.....	3
1. Renaming a folder does not save the change	3
2. Folders disappearing from Folders and Forward pages.....	3
3. DICOM media upload failures aren't notifying the user	3
4. Report saving from Web Service operation fails.....	3

New Features and Enhancements

1. Version information change for PAAS

The paas patch name label on the Server Information's Version Information page has changed to desktop apps.

2. Change secure connection default to AUTO

The system default for the secure connection setting has been changed to AUTO, meaning use secure communications if the source device is not listed in the local IP table as a LAN-accessible address.

3. Restore missing disk usage features

Some v7 disk usage features are now available in v8, including updating the disk usage information, cleaning unnecessary data from disk, and validating the repository configuration. These tools are on the Server Info/Disk Usage page.

Resolved Issues

1. Renaming a folder does not save the change

Renaming old folders failed because the check for an existing folder with the same name referenced the wrong field. This has been resolved.

2. Folders disappearing from Folders and Forward pages

If a user with a custom default worklist filter created a folder, it disappeared from the folders and forward page after an apache restart because all folders with list description tags are moved to the list section of the user's configuration file during restart. Note that the folders need to be recreated, but the content in those folders is unaffected.

3. DICOM media upload failures aren't notifying the user

When importing data from DICOM media, failure to create a DICOMDIR file due to invalid data on the media resulted in a silent failure. Now the user receives a notification indicating the media could not be imported.

4. Report saving from Web Service operation fails

Web services commands to edit a report issued to a worklist server in a multi-level dotcom failed because the RMI command sent to the child server did not use the RMI-specific session mechanism, resulting in permission failures. This has been resolved.