



# **Customer Release Notes**

**eRAD PACS  
Version 8.0  
Server Build 30  
Release Date June 3, 2021**

## Table of Contents

<b>New Features and Enhancements.....</b>	<b>3</b>
1. <b>Support for v7 HTTP API commands - import DICOM media .....</b>	<b>3</b>
2. <b>Restrict devices list when forwarding .....</b>	<b>3</b>
<b>Resolved Issues.....</b>	<b>3</b>
1. <b>Batch select feature missing the select all option.....</b>	<b>3</b>
2. <b>Batch select all feature only affects visible records on User Accounts and Groups page .....</b>	<b>3</b>
3. <b>Secure and not-secure sessionid cookies collide .....</b>	<b>3</b>
4. <b>NullPointerException during copy of hanging protocol to user profile .....</b>	<b>3</b>
5. <b>Webviewer error while creating an attachment .....</b>	<b>4</b>
6. <b>Exporting DICOM media license is not checked when creating Dicom Media .....</b>	<b>4</b>
7. <b>Other Lists page contains unidentified columns .....</b>	<b>4</b>

## New Features and Enhancements

### 1. Support for v7 HTTP API commands - import DICOM media

A command for importing data from DICOM media, `mediaImport.epw`, has been added to the web services library. Details are available in the web services programmer's manual. The default Mirth channel configuration has been updated to translate HTTP API `importMedia.jsp` commands to this web services command.

### 2. Restrict devices list when forwarding

The list of target devices available on the forward setup page, forward action setup page and the copy action page is configurable per-user. Configuration is available in the Target Device section of the user and group edit pages. Options include all devices, device accessible to the user's primary group, devices in a defined category, and the list of individual devices plus DICOM media, folders

## Resolved Issues

### 1. Batch select feature missing the select all option

The batch selection tool on some pages (Other Lists, User Accounts, Group Accounts) failed to select all studies. This has been resolved.

### 2. Batch select all feature only affects visible records on User Accounts and Groups page

The select all option in the batch select tool on the User Account and Group Account pages failed to collect all the matching entries, resulting in editing only some of the selected accounts. This has been resolved.

### 3. Secure and not-secure sessionid cookies collide

Due to a browser's secure cookie management, switching the system's security setting from HTTPS to HTTP prevented the user from establishing a new session (manifesting as login failures) until the browser removed or invalidated the secure cookie. This has been resolved.

### 4. NullPointerException during copy of hanging protocol to user profile

Copying a hanging protocol from one user to another user who has no hanging protocols defined, not even the built-in defaults, resulted in a crash because the system failed to handle the empty list. This has been resolved.

**5. Webviewer error while creating an attachment**

If the parent server is using HTTPS and the child is configured to use HTTP, requests to upload attachments failed due to a mixed context error. This has been resolved.

**6. Exporting DICOM media license is not checked when creating Dicom Media**

The DICOM media target appeared on the Forward setup page, and was active, when the user did not have permission to create DICOM media. This has been resolved.

**7. Other Lists page contains unidentified columns**

Two internal fields were not included in a hidden field list and showed up on the Other Lists table, unlabeled and uneditable. This has been resolved.