



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 39

Release Date October 13, 2021


Table of Contents

New Features and Enhancements.....	3
1. Query Qualifier prompt missing system suggested filters.....	3
2. Default licenses for plugin modules.....	3
3. Merging studies from different hubs	3
Resolved Issues.....	3
1. Add filter options missing on list pages	3
2. Folders cannot be seen on the GUI if the user never logged in	3
3. Users do not have the option to exit out of the archive retrieve window.....	4
4. The label for displayed time zone has one too many words "time" in the label.....	4
5. Multiple filters of the same field can be added to filter panel with context menu	4
6. On the Patient Folder page, the Forward button is disabled	4
7. Embedding DT type fields in notifications broken	4

New Features and Enhancements

1. Query Qualifier prompt missing system suggested filters

The query qualifier panel contains details about the poor query criteria and for users with restricted query rights, suggestions and interactive tools for improving them, based on the database configuration.

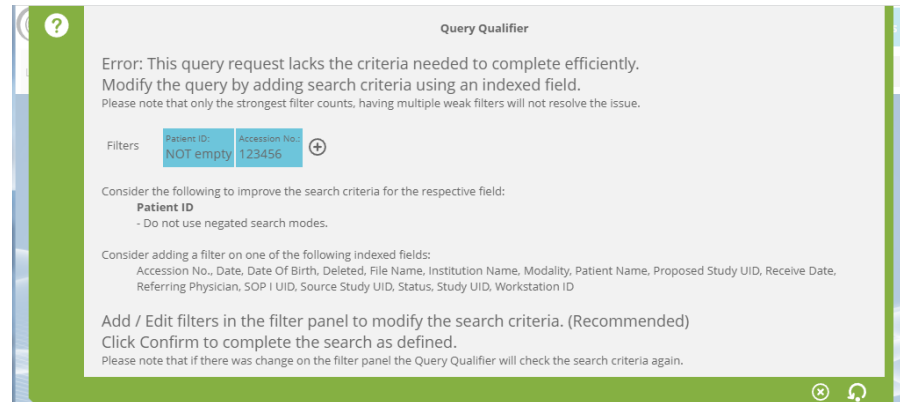

2. Default licenses for plugin modules

The base server license includes default licenses for the following plugin modules: MPR, 3D volume rendering, AVI, mask subtraction, stitching, WMI interface support, XML file drop and Radar CTRM. As a result, individual licenses for these plugin modules are ignored and each plugin is enabled for all users by default. Plugin modules not listed here still require plugin licenses. Existing plugin licenses, if needed, are still recognized, and enforced.

3. Merging studies from different hubs

When a user attempts to merge multiple studies, the system assures they exist on the same study before proceeding. If they are not on the same server, the message suggests the user use the Move tool to relocate the data first.

Resolved Issues

1. Add filter options missing on list pages

List pages failed to show the available fields from the filter panel for users without layout edit permission. This has been resolved.

2. Folders cannot be seen on the GUI if the user never logged in

If a user account exists but has never been used to log in, the account is uninitialized, causing a problem collecting folders. As a result, the Folders page would show no folders, for any users. This has been resolved.

3. Users do not have the option to exit out of the archive retrieve window

The option to cancel an archive move request was missing. It has been added to the curtain (page).

4. The label for displayed time zone has one too many words "time" in the label

A confusing label for the displayed time zone setting has been clarified.

5. Multiple filters of the same field can be added to filter panel with context menu

The system allows each field to exist only once as a worklist filter but users were not prevented from adding the same field multiple times. Now when a user attempts to add a field that is already specified, focus goes to the existing item.

6. On the Patient Folder page, the Forward button is disabled

During a recent change to protect against forwarding orders unnecessarily, the forward tool was disabled in the toolbar available from the worklist's patient folder. It has been restored.

7. Embedding DT type fields in notifications broken

Date-time fields, such as DATE, inserted into notification templates displayed an empty value because the feature failed to handle compound fields. This has been resolved.