



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 50
Release Date April 13, 2022

Table of Contents

New Features and Enhancements.....	3
1. Primary worklist percent loaded prior status	3
2. Make the rights setting color more visible when using dark mode	3
Resolved Issues.....	3
1. Timestamp on note incorrect if time is 12:XX:XX	3
2. CWL does not allow merging a study to an order	3

New Features and Enhancements

1. Primary worklist percent loaded prior status

New worklist fields exist to display the prefetch status of a study's priors. The column Percent Loaded Relevant indicates the prefetch status of the study and all priors matching the server's relevant priors configuration rules. The column Percent Loaded Priors indicates the prefetch status of the study and all matching priors. Note that the existing column's label has changed to Percent Loaded Current. These columns are updated if the study's Study Date is within the configured period, which defaults to 14 days. This setting is available at Server Configuration/Priors/Collect cache state for prior studies new than.

2. Make the rights setting color more visible when using dark mode

The background color of the individual rights fields when using the dark theme has been modified to make the setting indicator more visible.

Resolved Issues

1. Timestamp on note incorrect if time is 12:XX:XX

Parsing the date-time values in patient folder notes assumed a 12-hour clock rather than a 24-hour clock.

2. CWL does not allow merging a study to an order

Correcting a study to an order might fail because orders don't have an owner hub, which is required to determine where the combined study resides. As a result, manual corrections from the GUI reported a failure to the user. This has been resolved.