



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 52
Release Date May 13, 2022

Table of Contents

New Features and Enhancements.....	3
1. Admin GUI feature to review and delete nuked study files.....	3
2. Updated factory default "apply to current content" in action lists.....	3
3. Create viewer profile backup file after editing profile from the desktop viewer.....	3
Resolved Issues.....	3
1. Viewer sessions not recognized.....	3
2. Unable to break system lock from the worklist	3
3. Query Qualifier doesn't properly handle PDOB quick filter.....	3
4. Webviewer does not start if custom http port is used	3

New Features and Enhancements

1. Admin GUI feature to review and delete nuked study files

Nuked study files support study data which is used to populate a new web page for reviewing and deleting these files. The *Study Cleanup* page is available to users with **Support rights** from the *Admin* menu. The page is empty by default. Enter criteria to display a list of up to 5,000 nuked studies. The tools are consistent with those on the Worklist page. When cleaning studies that exist on child servers, start with the child servers before cleaning up the parent. Cleanup requests and results are logged in the forever log.

2. Updated factory default "apply to current content" in action lists

The default for the Apply to Current Content setting for all actions has changed to "No". Existing actions are not affected as long as they remain enabled. Once disabled, the new default shall be used when re-enabled, unless manually overridden during setup.

3. Create viewer profile backup file after editing profile from the desktop viewer

When the user updates their viewer settings, the existing profile file is saved as a backup so it can be restored later, if necessary. These backup files are propagated throughout the dotcom.

Resolved Issues

1. Viewer sessions not recognized

Viewer sessions were not recognized after restarting Apache, causing the cached data fields (eg, Percent Loaded) to report no data until the user logged out and back in. This has been resolved.

2. Unable to break system lock from the worklist

A system lock failed to be released because of a missing constructor. The constructor has been added to avoid the stuck lock. Also, when a user attempted to break the system lock, which is not permitted, the user received no explanation of why the lock remained. Now the user is informed he has no permissions to break a system lock.

3. Query Qualifier doesn't properly handle PDOB quick filter

Searching a worklist using a date value in the quick filter field that would result in a query qualifier exception returned an error message and no data because the date filter was improperly encoded in the database search request.

4. Webviewer does not start if custom http port is used

When using a custom port to launch the web viewer, the server would mishandle parsing the URL to locate the host name, causing the request to fail. This has been resolved.