



# **Customer Release Notes**

eRAD PACS  
Version 8.0  
Server Build 60

Release Date December 30, 2022

## Table of Contents

<b>New Features and Enhancements.....</b>	<b>3</b>
<b>1. Inherited user preferences/settings - GUI configuration.....</b>	<b>3</b>
<b>Resolved Issues.....</b>	<b>3</b>
<b>1. Global restriction for Worklist also applies to Logs page.....</b>	<b>3</b>
<b>2. Sending delete notification to WS devices is not working .....</b>	<b>3</b>

## New Features and Enhancements

### 1. Inherited user preferences/settings - GUI configuration

Group and system default settings are configurable from the GUI. The configuration page is accessible from the *Preferences* section of the *Admin/Server Settings/Web Server* page. Select the source account and then define one or more target accounts. Assign settings by checking the box in the settings section. Only checked settings will be copied to the target account(s). Use the search field to find a specific setting. (The section will be expanded.) Click Toggle Summary to review the changes to apply. Click Confirm to apply the changes. When done and after applying changes, close the panel by clicking the Cancel button in the bottom, right corner.

## Resolved Issues

### 1. Global restriction for Worklist also applies to Logs page

Global restrictions were unintentionally blocking non-study related data from the Logs page. This has been resolved.

### 2. Sending delete notification to WS devices is not working

The setting to send a notification to a web services device when a study is deleted was not getting saved when edited from the Devices page. The wrong event handler was incorrectly applied. This has been resolved.