



Customer Release Notes

eRAD PACS
Version 8.0
Server Build 61

Release Date January 10, 2023

Table of Contents

New Features and Enhancements.....	3
1. Improve resending to merged/split studies.....	3
Resolved Issues.....	3
1. Changing viewer version setting requires a manual page refresh	3
2. Resending into a chain of moveto flags fails to transfer all objects.....	3

New Features and Enhancements

1. Improve resending to merged/split studies

To improve the registration of objects re-sent after they have been deleted, the moveLateObjects task checks the study's resend flag when the target directory does not exist and when found, continues registering the object (rather than dropping it as a nuked object).

Resolved Issues

1. Changing viewer version setting requires a manual page refresh

When the user changes some settings, a session refresh updates those settings so they take effect immediately. Changing the assigned viewer was missing from this list of settings. As a result, changes to the applicable viewer didn't occur until the user initiated a new web session.

2. Resending into a chain of moveto flags fails to transfer all objects

Successive merges create a series of moveto flags so late arriving objects end up in the appropriate merged study. After deleting the merged study, resends of all but the last merged study failed to register due to mishandling the study identifier. This has been resolved.