



# Customer Release Notes

eRAD PACS  
Version 8.0  
Viewer Build 99

Release Date November 9, 2023

## Table of Contents

<b>New Features and Enhancements.....</b>	<b>3</b>
<b>1. Remove Enhanced ROI tool from viewer.....</b>	<b>3</b>
<b>Resolved Issues.....</b>	<b>3</b>
<b>1. PS upload displays a Completed dialog box even though the upload fails .....</b>	<b>3</b>
<b>2. Question mark displayed in managed back-up profile window .....</b>	<b>3</b>

## New Features and Enhancements

### 1. Remove Enhanced ROI tool from viewer

The enhanced ROI tool used for mammography images has been disabled and removed from the toolbar.

## Resolved Issues

### 1. PS upload displays a Completed dialog box even though the upload fails

**REVERSIBILITY NOTICE:** If there are reports saved locally, due to a bookmarked study or crashed viewer session, these cannot be recovered after reverting to a prior version.

The viewer failed to check for an error after it submitted presentation states to the server and always reported the send completed successfully. The viewer now checks the status returned by the server. When there are errors, the message indicates which presentation states failed. The user can decide to continue without them or to delete the presentation state in the viewer and recreate it.

### 2. Question mark displayed in managed back-up profile window

When using some language settings, the backup management panel failed to convert some characters to UTF-8 properly, resulting in invalid characters in place of accented characters in labels, dates and other text details.