



# Customer Release Notes

eRAD PACS  
Version 8.0  
Viewer Build 96  
Release Date August 11, 2023

## Table of Contents

<b>New Features and Enhancements.....</b>	<b>3</b>
<b>Resolved Issues.....</b>	<b>3</b>
<b>1. Double prefetching when WMI prefetching is configured .....</b>	<b>3</b>
<b>2. Viewer not closed properly when switching version .....</b>	<b>3</b>
<b>3. Viewer deadlock.....</b>	<b>3</b>

## New Features and Enhancements

### Resolved Issues

**DEPENDENCY NOTICE:** Dependencies exist. See details below.

**1. Double prefetching when WMI prefetching is configured**

If the prefetch service's configuration settings declared a prefetch server and a WMI client employed WMI prefetch, two prefetch tasks were activated. Studies included in the first request but not in the second request were not prioritized correctly and might not be handled as expected. This has been resolved.

**2. Viewer not closed properly when switching version**

**DEPENDENCY NOTICE: This change requires desktopApps-8.0.34 or later, but the viewer remains compatible with older desktopApp builds.**

(Evo8 viewer only) If the user's viewer version setting changes during an active viewer session, and the user initiates a new viewer session from the PACS worklist, the viewer could drop unsent data, including reports and presentation states, when closing the initial session.

**3. Viewer deadlock**

(Evo7 viewer only) When a user starts a viewer session while the prefetch Service is actively downloading the data included in the session, a deadlock might occur when the viewer takes over the prefetch pump. This has been resolved.