



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 65

Release Date August 19, 2021

Table of Contents

New Features and Enhancements	3
1. Change 3D angle tool menu label	3
2. Viewer streaming logging and info enhancements	3
Resolved Issues	3
1. Viewer hangs after disconnecting from the stream server	3
2. Viewer crash when two decompressions were applied on compressed data	3

New Features and Enhancements

1. Change 3D angle tool menu label

The label used for the 3D angle menu item and toggle button has changed to “3D Angle” and “Toggle 3D Angle”, respectively. The angle feature itself remains unchanged.

2. Viewer streaming logging and info enhancements

The streaming connection panel has some presentation anomalies, including the size of the panel, the order of the most-recently-used stream, and the override port number. Additional changes include combining errors and warning details on a single line and creating log entries using the correct port number.

Resolved Issues

1. Viewer hangs after disconnecting from the stream server

In a multi-server dotcom environment in which the viewer uses streaming to download data from multiple servers at the same time, a closed thread releases a shared handle but in some cases it was done in the wrong order, causing the viewer to hang. This has been resolved.

2. Viewer crash when two decompressions were applied on compressed data

If additional compressed data arrives at the same time the last data decompression task finishes, the viewer could mishandle the release of the decompression tasks causing the viewer to crash. This has been resolved.