



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 302

Release Date February 29, 2024



Table of Contents

New Features and Enhancements.....	3
1. Add a "Next Series" option to viewer	3
Resolved Issues.....	3
1. Smart Fit not effective for certain tomo series	3

New Features and Enhancements

1. Add a "Next Series" option to viewer

The viewer supports a quick method for loading the next or previous series into an image frame. The viewer command *Load Next Series* replaces the series in the selected image frame with the next one from the thumbnail panel. The viewer command *Load Previous Series* does the same in the reverse order.

Note: that if the thumbnail panel uses grouped series, the thumbnail group is traversed before moving onto the next/previous series or series group. If no image frame is selected, or if the selected frame contains a stack (meaning more than one series of data), the tool is disabled.

By default, the tool is assigned to no keyboard shortcut and no toolbar. Configuration is required.

Resolved Issues

1. Smart Fit not effective for certain tomo series

Due to a long-standing, now obsolete display requirement, thin images (i.e., images with few columns) displayed in wide frames might not be justified to the frame edge when indicated, as when Smart Fit justifies the chest wall in mammography images to the frame edge. This restriction has been eliminated, allowing images of any width to be moved, manually or automatically, to any position in the image frame.