



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 66

Release Date September 4, 2021

Table of Contents

New Features and Enhancements.....	3
1. Increase precision of (linear) measurements	3
2. Prioritize the supported CAD	3
3. Non-technical messages for CD upload error	3
4. Lock released message notification timeout	3
Resolved Issues.....	3
1. Crash due to print window.....	3
2. Disruptive prefetch errors messages	3

New Features and Enhancements

1. Increase precision of (linear) measurements

The precision of measurement values resulting from the viewer's measurement tools has been changed to display at least three significant digits, regardless of the applied units. Measurement data defined by the modality and included in the data object is unaffected by this enhancement.

2. Prioritize the supported CAD

The CAD list order setting is included in the default profile and can be copied to new and existing user accounts.

3. Non-technical messages for CD upload error

Details explaining a failure to import DICOM media come from the server and are included in the message displayed on the screen.

4. Lock released message notification timeout

The message informing the user that a lost lock has been released by the other user has changed to a passive notice that disappears after five seconds, without user intervention.

Resolved Issues

1. Crash due to print window

If the viewer session times out while a print panel is open, the viewer would crash when the user returns and logs in again. This has been resolved.

2. Disruptive prefetch errors messages

Transient prefetch errors resulted in a popup error message, interrupting the user's work. These errors are logged instead and not displayed in popup windows.