



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 72

Release Date October 30, 2021

Table of Contents

New Features and Enhancements.....	3
1. Progression view overlapping.....	3
2. Support cw3 in HTTP[s] channels	3
3. Support reconstruction of smaller image from cw3 data stream.....	3
Resolved Issues.....	3
1. Special characters not displayed properly	3

New Features and Enhancements

1. Progression view overlapping

The overlap indicators when using the progression view tool can be hidden by setting the configuration setting Options/Images/Progression/Show overlapped area to off.

2. Support cw3 in HTTP[s] channels

DEPENDENCY NOTICE: This feature requires SRV-729.

The key thumbnail (blue-line) data and HTTP/S frame data (as opposed to streaming frame data) can be requested from cw3 files, avoiding the need to process cw1 files if cw3 files already exist.

3. Support reconstruction of smaller image from cw3 data stream

DEPENDENCY NOTICE: This feature requires SRV-729.

Decompressing the initial quality images for use in the thumbnail panel has been optimized to allow the request to produce a smaller image.

Resolved Issues

1. Special characters not displayed properly

Some special characters used in names and rendered by the viewer, primarily on tabs and in the DICOM information panel, were displayed incorrectly. This has been resolved.