



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 73

Release Date November 10, 2021

Table of Contents

Resolved Issues.....	3
1. Monitor settings end up in an invalid state if the primary monitor is unplugged	3
2. Unlinking the master magic glass' image doesn't remove its followers	3
3. Shortcut manager window displays two edit buttons.....	3
4. Timer of downloaded IQ image percentage indicator not stopped.....	3
5. Missing optimization for lossy images in cw3.....	3

Resolved Issues

1. **Monitor settings end up in an invalid state if the primary monitor is unplugged**

If the primary monitor number is greater than the number of available monitors, which can happen after disconnecting monitors after assigning the primary monitor, no primary monitor is defined and some of the viewer's layout functions fail. In this state, the viewer automatically reassigned the primary number to the highest available monitor number.

2. **Unlinking the master magic glass' image doesn't remove its followers**

When magic glass linking is active and multiple magic glass panels appear over each linked image frame, unlinking an image frame left the linked magic glass panel visible and unresponsive.

3. **Shortcut manager window displays two edit buttons**

The shortcut manager panel labeled the Delete button (for user-defined macros) incorrectly. This has been resolved.

4. **Timer of downloaded IQ image percentage indicator not stopped**

When configured to use hyper streaming, the timer used to refresh the IQ image download percentage was not stopped once the download completed.

5. **Missing optimization for lossy images in cw3**

An image arithmetic optimization applied to lossy images was incorrectly omitted when the software was merged with the server's equivalent code, causing slower decompression speeds. This has been resolved.