



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 74

Release Date December 8, 2021


Table of Contents

New Features and Enhancements.....	3
1. Display CE Mark in viewer.....	3
Resolved Issues.....	3
1. Window/Level doesn't work with LINEAR_EXACT VOI LUT function.....	3
2. Stack ruler wrong with rotated RMLO images	3
3. Exported Plugin Series Not Listed in Thumbnail Panel	3
4. Cw3 reconstruction fails when the resampled image has one dimension less than 64 pixels..	3
5. Next/Previous study sometimes causes Invalid study error.....	3

New Features and Enhancements

1. Display CE Mark in viewer

The CE Mark is displayed on the viewer's About panel.

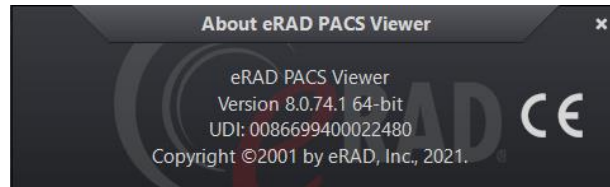

Resolved Issues

1. Window/Level doesn't work with LINEAR_EXACT VOI LUT function

If an object declared the image displays using a LINEAR_EXACT VOI LUT function, window and leveling failed to work because the viewer already overwrote the user-specified values with the full range values. This has been resolved.

2. Stack ruler wrong with rotated RMLO images

In some conditions, rotating a mammography image 90 degrees inverted the stack ruler's direction. This has been resolved.

3. Exported Plugin Series Not Listed in Thumbnail Panel

A change in build 70 introduced a refresh problem which failed to display images exported from a plugin module in the thumbnail panel. This has been resolved.

4. Cw3 reconstruction fails when the resampled image has one dimension less than 64 pixels

Image reconstruction failed an internal consistency check when the resampled image had a dimension less than that used by the thumbnail images (64 pixels). This has been resolved.

5. Next/Previous study sometimes causes Invalid study error

A timing issue could cause the viewer to read the next or previous study URL before it was ready, resulting in an invalid study error message. This has been resolved.