



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 84
Release Date July 14, 2022

Table of Contents

New Features and Enhancements.....	3
1. Prefetch CW3 (Hyper Streaming) data	3
Resolved Issues.....	3
1. Commands are not shown properly when filtering in Commands column.....	3
2. "New line" button disabled for context menus.....	3
3. Scanning from feeder doesn't read multiple pages.....	3
4. Makespace upon Service start when disk space is low	3

New Features and Enhancements

1. Prefetch CW3 (Hyper Streaming) data

DEPENDENCY NOTICE: Requires **8.0.41** installed on the server.

If a server has only cw3 files available when it receives a prefetch request from a viewer that supports cw3 files, it sends the cw3 files rather than generate cw1 or raw files.

Resolved Issues

1. Commands are not shown properly when filtering in Commands column

When searching for commands in the toolbar customization panel, matching values can show up as empty strings or the scroll bar might be missing, until the user forces the GUI to refresh. This has been resolved.

2. "New line" button disabled for context menus

When editing the toolbar configuration, the new line button was improperly disabled for context menus, preventing users from adding new entries to context menus. This has been resolved.

3. Scanning from feeder doesn't read multiple pages

For some scanning devices, the document scanning tool omitted a required property in the command and only the first page would be scanned. This has been resolved.

4. Makespace upon Service start when disk space is low

When initializing the caching service, before the viewer is running and the cache size has been defined by the configuration, the service could delete studies if the total free space on the workstation exceeds the size of the next study being initialized. This has been resolved.