



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 85
Release Date August 13, 2022

Table of Contents

New Features and Enhancements.....	3
1. DCMTK update for the viewer.....	3
2. Apply new DCMTK to Overlays and DICOM Information	3
3. Apply new DCMTK to Presentation States	3
4. Apply new DCMTK to Mammo CAD SR objects.....	3
Resolved Issues.....	3
1. Aidence AI Character Error in Overlay	3

New Features and Enhancements

1. DCMTK update for the viewer

The viewer's DCMTK library has been updated to version 3.6.7. Some performance improvements have been applied as well.

2. Apply new DCMTK to Overlays and DICOM Information

Overlays (DICOM Information) make use of the updated DCMTK toolkit, adding support for text values using an alternative character set.

3. Apply new DCMTK to Presentation States

Presentation states, including key images, make use of the updated DCMTK toolkit, adding support for text values using an alternative character set.

4. Apply new DCMTK to Mammo CAD SR objects

CAD overlays make use of the updated DCMTK toolkit, adding support for text values using an alternative character set.

Resolved Issues

1. Aidence AI Character Error in Overlay

Presentation states containing text strings using alternative character sets, ie, not latin-1, or special font characteristics, such as superscript, were presented incorrectly because the viewer's DCMTK tools did not support alternative character sets. The DCMTK toolkit has been updated.