



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 88

Release Date November 23, 2022

Table of Contents

Resolved Issues.....	3
1. When the cache is full both prefetch and Makespace stopped	3
2. Mammo CAD isn't shown in hanging protocol.....	3
3. Interrupted prefetch uses invalid host name to continue study prefetching.....	3

Resolved Issues

1. When the cache is full both prefetch and Makespace stopped

When the viewer's cache became full, the Makespace task stopped prefetching to avoid a continuous load cycle. When the prefetch list changed it could not load new studies or delete old ones. This has been resolved.

2. Mammo CAD isn't shown in hanging protocol

The fix for missing CAD markers when restoring a bookmarked viewer session disabled automatic application of CAD markers for stored data. As a result, the viewer failed to recognize the default CAD display setting saved in hanging protocols. This has been resolved.

3. Interrupted prefetch uses invalid host name to continue study prefetching

When prefetching a study is interrupted, it is retried during the next prefetch cycle. But the server's hostname was saved incorrectly, causing the retry to fail. As a result, the images would have to download on-the-fly when the study is opened. This has been resolved.