



Customer Release Notes

eRAD PACS
Version 8.0
Viewer Build 92
Release Date March 8, 2023

Table of Contents

New Features and Enhancements.....	3
1. Username and password fields for XML Interface Speech Recognition.....	3
Resolved Issues.....	3
1. Prefetch list sorting mechanism can add used study to unused list	3

New Features and Enhancements

1. Username and password fields for XML Interface Speech Recognition

The ability to specify the user ID and password included in the XML file dropped when one is dropped has been extended to all reporting options, including external reporting systems configured using the generic XML file drop. The configuration fields are available in the External Reporting panel in the viewer settings tool.

Resolved Issues

1. Prefetch list sorting mechanism can add used study to unused list

If the cache is full and the study open in the viewer was prefetched, reordering the list of prefetched studies (to determine which are unused and therefore eligible for removal from the cache) might put the open study on the unused list, resulting in a crash or lost connection between the viewer and cache manager. This has been resolved.